

audionexx

USER GUIDE

Hearing Device



♪ We make hearing easier ♪

Get ready to start hearing everything better

GET GOING IN 4 EASY STEPS

Step 1

Pick your dome size

Step 2

Turn on device

Step 3

Find the right setting

Step 4

Cleaning your device

Features



- ① Removal handle
- ② Charging contacts
- ③ Volume control
- ④ Microphone inlet
- ⑤ Power switch
- ⑥ dome

Find the perfect fit

CHOOSING THE HEADER

These domes provide a much more comfortable wearing experience and significantly reduce or eliminate the stuffy, plugged sensation of closed fitted ear plugs.



Tip: If the Regular size dome feels too loose, swap with the large size. If powerful amplification is needed, swap the open dome with the closed dome. This will enable you to raise the volume without creating any whistling/feedback

CHANGING THE DOME

The dome is an earpiece used to secure the hearing aid comfortably in your ear canal and to protect the sound outlet from earwax intrusion.

Put the dome over the grooves on the adapter to mount it.



Wear

- ① Hold the hearing aid with the removable handle at the bottom and the trimmer at the top between your thumb and forefingers.
- ② Gently insert the canal part of the hearing aid into your ear by pushing back and tucking the hearing aid completely into the right place with your forefinger.



Power



- ① Hearing Device Charging Inserts
- ② Left/right Indicators



- ③ Power Indicator
- ④ Type-C Connection

PREPARING THE CHARGER

Connecting the charger to a power supply

- a) Plug the charging cable into the power supply.
- b) Plug the smaller end into the USB port of the charger.
- c) The power indicator light will glow blue when the charger is connected to the power supply.



CHARGING THE HEARING DEVICES

- Before using your hearing device for the first time, it is recommended that you charge it for 6 hours.
- The hearing device contains a built-in and non-removable rechargeable battery.
- Turn off the hearing device before charging
- The hearing device must be dry before charging.
- Power off the hearing device before charging.
- Please ensure to charge and use the hearing device within operating temperature range: +5° to +40° Celsius (41° to 104° Fahrenheit).

When the device is placed on the charger, the indicator light flashes for 3 seconds and starts charging.

When charging, the indicator light will remain red.

When 100% charged the light switches to green. When the devices are full, the hearing device indicator light will remain green.

Red light = 0-99% charging
Green light = 100%



Adjust

AMPLIFICATION LEVELS

Adjust the amount of output sound, turn the trimmer clockwise for max sound amplification.

Note : Handle the button gently, the operation requires only a minimum of force. If there is any resistance when rotating the button, just stop. (do not pass 270°)



Right = raise volume
Left = lowers volume

CLEANING BRUSH TOOL


Use the wire loop on the cleaning brush to adjust the amount of output sound, turn the trimmer clockwise for max sound amplification.



Maintenance

REPLACE THE DOME

Replace the dome of your hearing instrument every week. This will help to prolong the lifetime of your instrument.

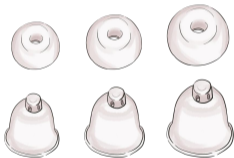

NOTE: Clean your hands thoroughly before replacing the Dome. Never change the Dome with wet or dirty hands.

Troubleshooting

No Sound	
Volume may be turned down	Turn volume control up
Battery may be low or dead	charge hearing aid
Hearing instrument may not be turned on	Turn on Hearing instrument
Start up delay is on	Wait a few seconds for device to turn on
Dome is blocked with moisture or debris	Use the vent cleaning tool to clear the dome

Sound Level is Too Low

Dome not inserted properly	Re-insert dome
Battery may be low or dead	Recharging
Dome is blocked	Clear the dome
Your hearing has changed	Consult your hearing healthcare professional
You may have excessive cerumen	Consult your hearing healthcare professional
Volume may be turned down	Rotate volume control wheel up

audionexx

Thank you for shopping with Audionexx!:)